

Biohazard Clean Up Kits

# Everything required in one simple pack.

Why make it complicated?



Efficient, simple stepped approach to Biohazard control. Each kit contains individual applications with everything required to deal with an incident, all in one simple pack.

**Simple to grab,  
Simple to use,  
Simple to destroy...**

**NOW AVAILABLE IN  
SMALL with  
2 APPLICATIONS**



# Everything required in one simple pack.

Kits contain individual applications as an efficient, simple stepped approach to biohazard control.




1.

Put on the protective apron and gloves.



2.

Sprinkle absorbent granules over spill.

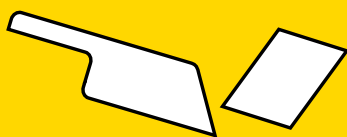

3.

Allow spill to turn gelatinous, then scoop and scrape it into biohazard bag.



4.

Disinfect the area, and wipe down. Place everything into the biohazard bag and destroy.

**Simple to grab,  
Simple to use,  
Simple to destroy...**

## BIOHAZARD BODY FLUID CLEAN-UP KIT IN INTEGRAL AURA BOX

Available as 2 or 5 application kit.  
Everything required to clean-up in one simple pack.  
Incident packs are individually boxed.  
Provides containment and disposal in each application.  
Meets current HSE requirements.

**SIZE SMALL** 22.5cmH x 23cmW x 9.5cmD  
**SIZE LARGE** 32cmH x 35cmW x 10cmD

### CONTENTS

#### SINGLE APPLICATION CONTAINS

- |                            |                     |
|----------------------------|---------------------|
| 1 Absorbent Granules (10g) | 1 Gloves (Pair)     |
| 1 Biohazard Bag            | 1 Pack Tissues (10) |
| 1 Disinfectant Spray (8ml) | 1 Polythene Apron   |
| 1 Disinfectant Wipe        | 1 Scoop Scraper     |
| 2 Dry Wipes                |                     |



### REF PRODUCT

2717	2 Application Body Fluid Clean-Up Kit
718	5 Application Body Fluid Clean-Up Kit
717	1 Application Body Fluid Clean-Up Kit Refill

## BIOHAZARD COMBINATION KIT IN LARGE INTEGRAL AURA BOX

Complete biohazard and sharps protection.  
Deals with both incident types.  
Safe disposal products included.  
Contains 3 clean-up, 2 sharps applications.  
Meets current HSE requirements.

**SIZE** 32cmH x 35cmW x 10cmD

### CONTENTS

#### SINGLE BODY FLUID APPLICATION CONTAINS

- 1 Absorbent Granules (10g)
- 1 Biohazard Bag
- 1 Disinfectant Spray (8ml)
- 1 Disinfectant Wipe
- 2 Dry Wipes
- 1 Gloves (Pair)
- 1 Pack Tissues (10)
- 1 Polythene Apron
- 1 Scoop Scraper

#### SINGLE SHARPS APPLICATION CONTAINS

- 1 Biohazard Bag
- 1 Disinfectant Spray (8ml)
- 1 Disinfectant Wipe
- 1 Forceps
- 1 Gloves (Pair)
- 1 Sharps Container 0.2ltr



### REF PRODUCT

983	5 Application Combination Clean-Up Kit
-----	--

## BIOHAZARD SHARPS CLEAN-UP KIT IN INTEGRAL AURA BOX

Handles up to 5 incidents.  
Contains handling and protective items.  
Packs include safe disposal products.  
Meets current HSE requirements.

**SIZE** 32cmH x 35cmW x 10cmD

### CONTENTS

#### SINGLE APPLICATION CONTAINS

- 1 Biohazard Bag
- 1 Disinfectant Spray (8ml)
- 1 Disinfectant Wipe
- 1 Forceps
- 1 Gloves (Pair)
- 1 Sharps Container 0.2ltr



### REF PRODUCT

988	5 Application Sharps Clean-Up Kit in Large Integral Aura Box
987	1 Application Kit Sharps Clean-Up Kit Refill

