

BurnSoothe®

soothing, cooling gel for burns

Single use dressings.

Various sizes to treat different body areas.

Sterile and non-toxic.

Instant minor burn relief and protection.

Available as dressings or bottled gels.



How to apply a BurnSoothe® dressing.

HEAT BURN

- Open BurnSoothe® sachet and lay the dressing across the wound site.
- Squeeze the remaining gel from the sachet across the dressing.
- Secure BurnSoothe® in place with a conforming bandage, taking care not to apply too much pressure.

Seek medical attention if necessary.

CHEMICAL BURN

- Wash the wound site continually in cold water for approximately 15-20 mins.
- Follow the instructions as per Heat Burn shown above.



BurnSoothe®

soothing, cooling gel for burns

- Sterile trauma hydrogel burn dressing.
- Minimises trauma and skin damage.
- Non adherent to burnt skin
- Cools and soothes.
- Non-toxic



BurnSoothe Burn Gel, Sachet
3.5g Sachet

REF	SIZE	UNIT	PACK
391	3.5g	Box of 25	1



BurnSoothe Burn Dressing
2.5cm x 5cm

REF	SIZE	UNIT	PACK
392	2.5cm x 5cm	Box of 10	1



BurnSoothe Burn Dressing
5cm x 15cm

REF	SIZE	UNIT	PACK
393	5cm x 15cm	Box of 15	1



BurnSoothe Burn Dressing
10cm x 10cm

REF	SIZE	UNIT	PACK
394	10cm x 10cm	Box of 10	1



BurnSoothe Burn Dressing
20cm x 20cm

REF	SIZE	UNIT	PACK
395	20cm x 20cm	Box of 5	1



BurnSoothe Burn Dressing
Face Mask

REF	SIZE	UNIT	PACK
396	Face Mask	Box of 5	1



BurnSoothe Burn Dressing
10cm x 40cm

REF	SIZE	UNIT	PACK
400	10cm x 40cm	Box of 5	1



BurnSoothe Burn Dressing
20cm x 45cm

REF	SIZE	UNIT	PACK
401	20cm x 45cm	Box of 1	1



BurnSoothe Burn Dressing
60cm x 40cm

REF	SIZE	UNIT	PACK
402	60cm x 40cm	Box of 1	1



BurnSoothe Burn Gel, Bottle
125ml Bottle

REF	SIZE	UNIT	PACK
399	125ml	Box of 6	1



BurnSoothe Burn Gel, Bottle
50ml Bottle

REF	SIZE	UNIT	PACK
2399	50ml	Box of 6	1

