

## HEAD IMMOBILISER INSTRUCTIONS FOR USE.



### FEATURES

- Unit head strap is designed to conform to patient's forehead.
- Helps the patient to hear the operator and allows operator to check the patient's ears.
- Minimal interference with X-Ray, MRI or CT procedures.
- Waterproof plastic coating makes it easy to prevent bacterial growth within components. Coating will not absorb blood or bodily fluids.
- Durable and easy to clean.
- Allows comfortable direct contact with skin.

### CHECK LIST

Each Unit should have:

- 1x Base Board with eight 'D' shape loops
- 2x Block with round hole
- 2x Immobilisation Belt
- 2x Head Strap

### MAINTENANCE

- Clean with wet cloth after use.
- Do not wash black Velcro parts with water.
- Keep dry and ventilated.



**1 USE THE CENTRAL TOP BELT TO ATTACH BASE BOARD.**

Place the base board onto the Spine Board (Velcro-side up), and insert the top belt through the central hole at the top of the Spine Board.



Pull the belt through the base board's D-loop. Reverse and stick on the belt's Velcro.



**2 SECURE THE BASE BOARD ON THE SPINE BOARD.**

Attach one side of the belt by feeding it through one of the two middle D-loops and securing to the belt's Velcro.



Repeat with the other middle belt and D-loop.



Turn over the Spine Board and immobilise the other side of the belts through the symmetrical D-loops.



**3 USE THE TWO BLOCKS TO IMMOBILISE THE PATIENT'S HEAD.**

Carefully place the patient onto the spine board, with the correct position for head immobilisation.



Attach a block to one side of the base board using the Velcro (with the straight side facing inward).



Attach the second block on the other side of the patient's head using the Velcro.



**4 SECURE THE TWO RETENTION STRAPS.**

Secure one strap between the top symmetrical D-loop using the Velcro to secure the patient's forehead.



Loop the remaining strap through the bottom D-loop to secure the patient's chin.



**The blocks should not be loose nor uncomfortably tight, but firmly applied.**

**WARNING**  
Follow the instructions exactly as described above and without any deviation.

