



DEPENDAPLAST Pilfer Proof Plasters

Once Removed, Ready to Apply



An opportunity to
**double your profit on
dispensable plaster refills.**

For many years, the high cost price of plasters in Pilfer Proof wallets has seriously dented the sales potential for this range of products. Reliance medical now offer this product at breakthrough prices.



Make the comparison and double your sales profit.

The Dependaplast Plaster Dispenser can be operated by the casualty with a single-handed procedure. The plasters are dispensed downwards, allowing easy application without effecting sterility.



STEP 1.
Secure the Plaster Dispenser to the wall and stock with two wallets.



STEP 2.
Select the plaster of choice by holding and pulling on the top tab.



STEP 3.
Release the plaster by pulling upwards.



STEP 4.
Apply the plaster to the affected area.

REF	PRODUCT	SIZE	UNIT	PACK
907	Dependaplast Washproof Pilfer Proof Plasters	27 7.2cm x 1.9cm 18 7.2cm x 2.5cm	Box of 5	1
916	Dependaplast Fabric Pilfer Proof Plasters	24 7.2cm x 1.9cm 16 7.2cm x 2.5cm	Box of 5	1
984	Dependaplast Blue Pilfer Proof Plasters	21 7.2cm x 1.9cm 14 7.2cm x 2.5cm	Box of 5	1
3900	Dependaplast Release Pilfer Proof Plaster Dispenser, Empty	Fits 2 wallets, mix or match	1	1

